

# Standard Charger & Custom Charger



The Standard Charger operates as a portable charging station, dry-aid kit and case for standard lithium-ion rechargeable hearing aids. The Custom Charger operates as a portable charging station and case for custom lithium-ion rechargeable hearing aids. Both chargers are equipped with LED lights to indicate charging status for both the hearing aids and the on-board battery.

## LED Indicators for Hearing Aid Charging

The front of the charger is equipped with two LED indicators to provide information about hearing aid charging.

- When the hearing aids are placed into the charger, the front LED lights will glow green to indicate successful charging.
- When the hearing aids are fully charged, the front LED lights will be solid green indicating a full charge.
- If there is an issue with charging and the hearing aids are in a fault state, the front LED lights will blink red.

**NOTE:** If a fault state is detected, remove the hearing aids from the charger, wait until the LEDs turn off, and re-insert the hearing aids. If the fault state continues, contact your hearing professional.



## LED Indicators for On-board Battery

---

The back of the charger is equipped with four LED indicators to provide information about the status of the on-board battery. The on-board battery allows charging of the hearing aids to occur while the charger is away from a power source.

- Four solid green lights indicate the charger is plugged into a power source and is fully charged.  
**NOTE:** *When charging without the power cord, the back LEDs will turn off after 10 seconds to preserve battery life.*
- When away from a power source, four solid lights indicate the battery has more than 75% charge.
- Three solid lights indicate the battery has less than 75% charge.
- Two solid lights indicate the battery has less than 50% charge.
- One blinking light indicates that the charge is low, and the charger should be connected to a power source to charge the on-board battery.

**NOTE:** *When plugged into a power source, both the hearing aids and on-board battery will charge simultaneously.*



## Use

---

- 1 Standard hearing aids and on-board battery will be fully charged in 2.5 hours. Custom hearing aids will be fully charged in 3.5 hours.
- 2 Hearing aids can be charged up to 3 times via the on-board battery without being connected to a power source.
- 3 It is safe to keep the hearing aids in the charger as they cannot overcharge.
- 4 Charging will occur with the lid open or closed.  
**NOTE:** *It is recommended to keep the Standard Charger lid closed to preserve the desiccant puck if also using as a dehumidifier.*
- 5 Hearing aids automatically power on when removed from the charger and power off when inserted into the charger.
- 6 If the charger is not plugged in and runs out of charge, the hearing aids will turn ON in the charging case.